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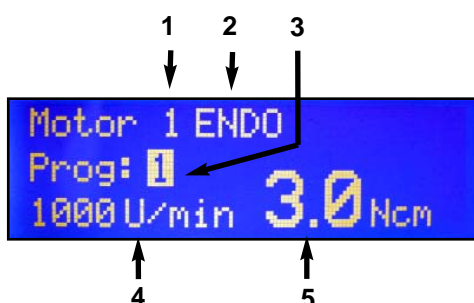
This mode is available only for the use with motors of the type NSK-NLX plus.

Description

The Endo-mode for motors can be used for different endodontic treatment purposes. It is possible to save the maximum rotation speed and the maximum motor torque for each single motor in 8 different programs.

The ATR (automatic reverse) function effects an automatic reversing of the motor rotation direction if reaching the set maximum motor torque.

The ATRF (automatic reverse - forward) function effects an automatic reversing of the motor rotation direction for approximately 1 second. After this time period the motor rotates again in its primary rotation direction.

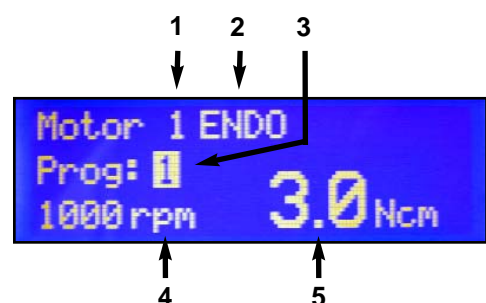
Operation

Take the desired motor from its holder or its tray.

Press briefly the Endo-key in order to switch on or switch off the Endo-function of the motor. At the display appear the following information:

- (1) Number of the active motor
- (2) Indication of the active Endo-mode
- (3) Current Endo-program
- (4) Maximum motor rotation speed in rpm
- (5) Maximum motor torque in Ncm

The factory made pre-set values for each program are 1000 rpm and 3,0 Ncm.

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Selecting program

Press briefly the "Star"-key in order to switch between the single programs.

Setting the maximum rotation speed

Press the "Plus/Minus"-key in order to set the value of maximum rotation speed. Selectable are values between 100 ~ 1000 rpm. The last displayed value is saved automatically.

↑
Maximum motor rotation speed

 The maximum rotation speed must comply with the manufacturer's instructions of the endodontic tool!

Setting the maximum motor torque

Press at first the "Endo"-key longer than 2 seconds. The value for the maximum motor torque blinks at the display.

Press the "Plus/Minus"-key to set the maximum value of motor torque. Selectable are values between 0,3 ~ 3,0 Ncm. The last displayed value is saved automatically.

↑
Maximum motor torque

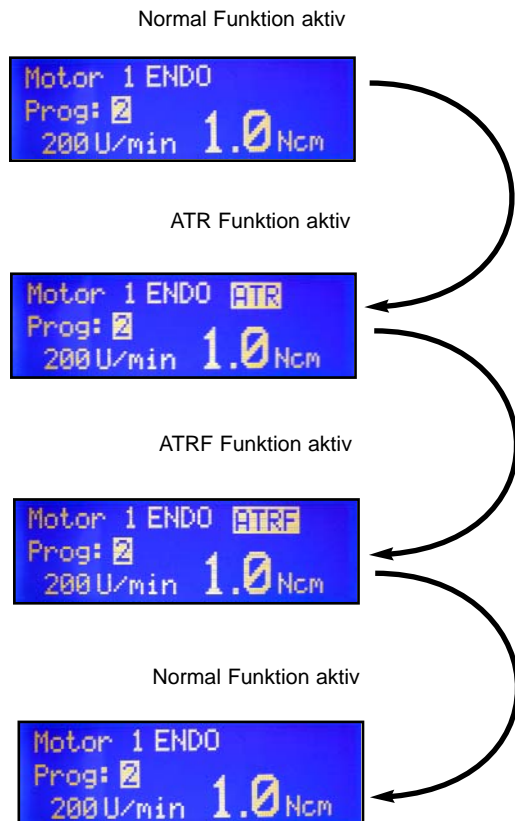
After setting the desired value, press briefly the "Endo"-key in order to end the setting mode.

 The maximum rotation speed must comply with the manufacturer's instructions of the endodontic tool!

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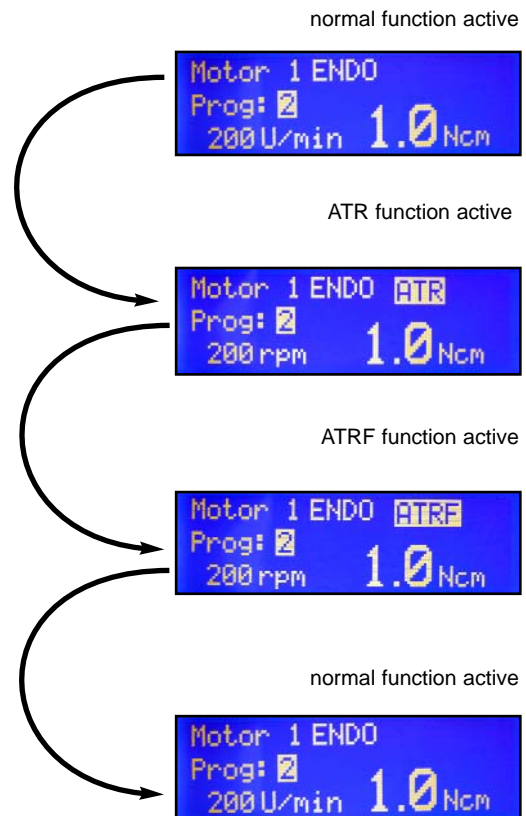
ATR (automatic reverse) / ATRF (automatic reverse - forward)

Press the "Rotation direction"-key longer than 2 seconds in order to switch between the functions "Normal/ATR/ATRF"
The display shows the active function.

With active ATR/ATRF function a permanent signal tone sounds during the automatic left-handed rotation of the motor.

The signal tone can be deactivated via the SoftDIP switch number3/Set3.

The second instrument status is not available during active Endo-mode.

With active ATR/ATRF function the left handed rotation of the motors can not be switched manually.